medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Hungary Kft.
Bokor u. 21
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Italia S.r.l.
Via Guido Rossa 28
40033 Casalecchio di Reno-Bologna
Italy
T +39 051 613 24 84
F +39 051 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

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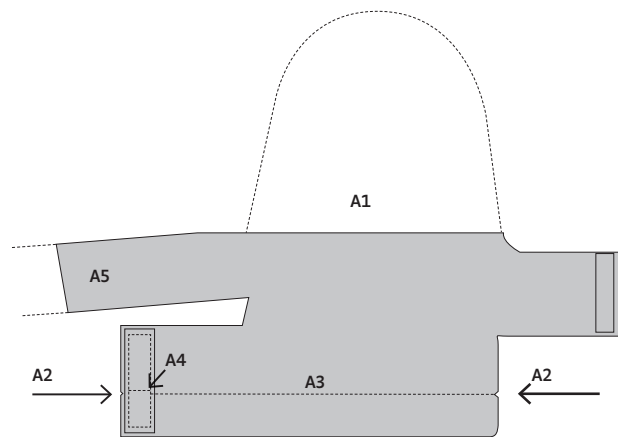
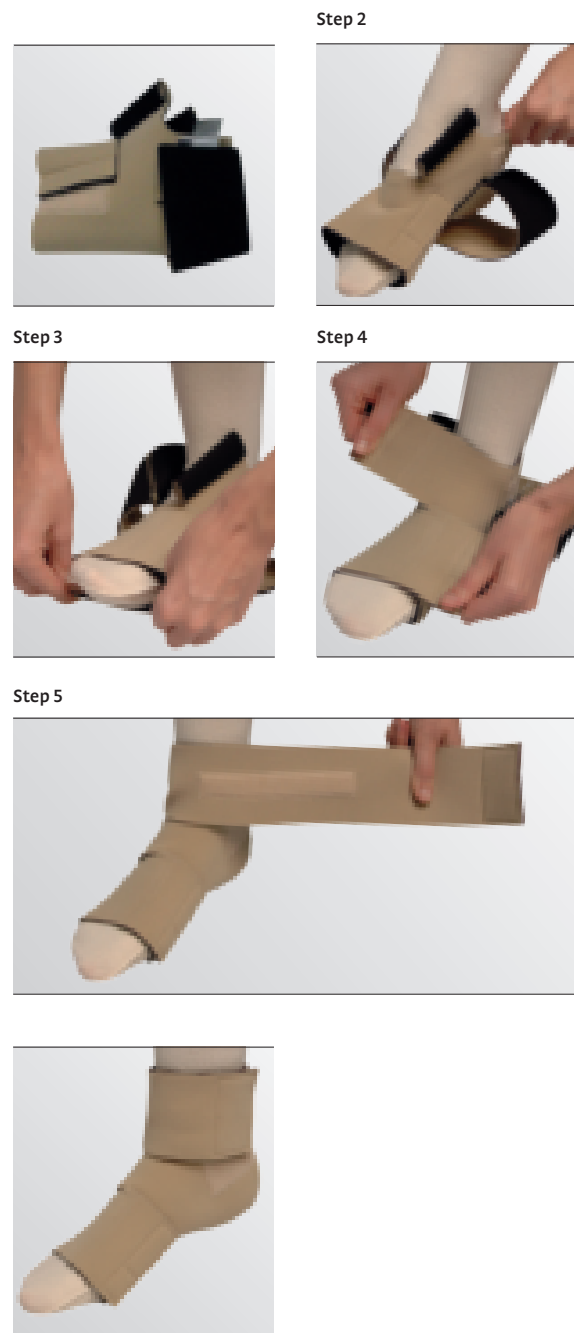
circaid juxtafit premium
ankle foot wrap

CIRCAID

circaid® juxtafit® premium
foot pieces
inelastic compression system

Instructions for use. Gebrauchsanweisung.
Инструкция по использованию. Instruções de uso.
Instrucciones de uso. Mode d'emploi. Gebruiksaanwijzing.
Istruzioni per l'uso. Bruksanvisning. Brugsvejledning.
Instrukcja użytkowania

medi. I feel better.

circaid® juxtafit® premium foot pieces

Purpose

The foot wrap is a single patient, reusable, non-invasive, non-sterile device designed to provide foot and ankle compression to patients with venous and lymphatic disorders. The wrap consists of a series of juxtaposed, inelastic bands extending from a central portion of the wrap. The wrap contours the foot and is designed to cover from the base of the toes to just above the ankle, overlapping with the legging.

Indications

- Lymphedema
- Other forms of edema
 - Venous
 - Post traumatic
 - Post surgical
 - Lipedema
- Varicose veins (all types)
- Chronic venous insufficiency (CVI)
- Venous stasis disease
- Venous valvular insufficiency (VVI)
- Venous insufficiency
- Post thrombotic syndrome
- Venous ulcer (stasis ulcer)
- Post sclerotherapy
- Venous eczema
- Lipodermatosclerosis

Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and

compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the leg
- Intolerance to compression material

Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. chronic venous insufficiency, etc.).

- Moderate and/or severe peripheral arterial disease
- Septic phlebitis
- Phlegmasia cerulea dolens
- Decompensated congestive heart failure
- Infection in the ankle and / or foot

Storage instructions

Store in a dry place

Keep out of sunlight

The shelf-life date is printed on the carton label with the symbol of a "sand glass". Made-to-measure garments are products for immediate use and do not have a shelf-life date.

Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O. BOX NO. 91464
DUBAI, UAE
T: +971 4 2557344
F: +971 4 2557399
md@taurusgroup.net

medi Polska Sp. z o. o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T+48 32 230 60 21
F+48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisboa
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
United Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center "Flora Park"
Kiev 04112
Ukraine
T +380 44 591 1163
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
T +1 336 4 49 44 40
F +1 888 5 70 45 54
info@mediusa.com
www.mediusa.com

circaid® juxtafit® premium foot pieces

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